

PoCUS ULTRASOUND MACHINES

INFECTION CONTROL



- Use damp Microfibre wipes ONLY
(NOT alcohol) to clean transducers, cords, console & machine
- Wipe off excess gel from transducers BEFORE using damp Microfibre cloth to clean
- Wipe machine console over with dry paper towel AFTER cleaning (water droplets may activate touch sensitive keys)
- Use sterile transducer covers for ALL procedures or in presence of fluids/blood/exposed body tissues

EQUIPMENT CARE

- Ensure machine is returned to designated parking area
with POWER CORD PLUGGED IN to charge backup battery
- Store transducers correctly in holders
- Do not drop or knock transducers
- Keep cords and cables off floor

PLEASE REPORT ALL FAULTS & DAMAGE IMMEDIATELY

Service engineer contact details & machine serial numbers are held by Department administrators

Other issues email pocusprogram@monashhealth.org